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The screenshot shows a medical report form for a CT scan. The title bar at the top reads "CT, MIGGOS LORETTA, 0516644, F, 2010-10-26 12:33:56, , CT CHEST W". The form is divided into several sections:

- 1**: Title bar area.
- 2**: "Related exam" button.
- 3**: Search filters including "방사선과" (Radiology), "*Any code", and "*Any description".
- 4**: "Update" button.
- 5**: "Save standard report" button.
- 6**: "Close" button in the bottom left.
- 7**: "Suspend" button in the bottom left.
- 8**: "Approve" button in the bottom right.

The main body of the form contains sections for "[Reading]" and "[Conclusion]", each with a text area for input. Below these are fields for "Creator", "Dictator", "Transcriber", "Approver 1", and "Approver 2". At the bottom, there are buttons for "None", "Close", "Suspend", "Print", "Merge", "Save", "Approve", "Messa...", and another "Close".

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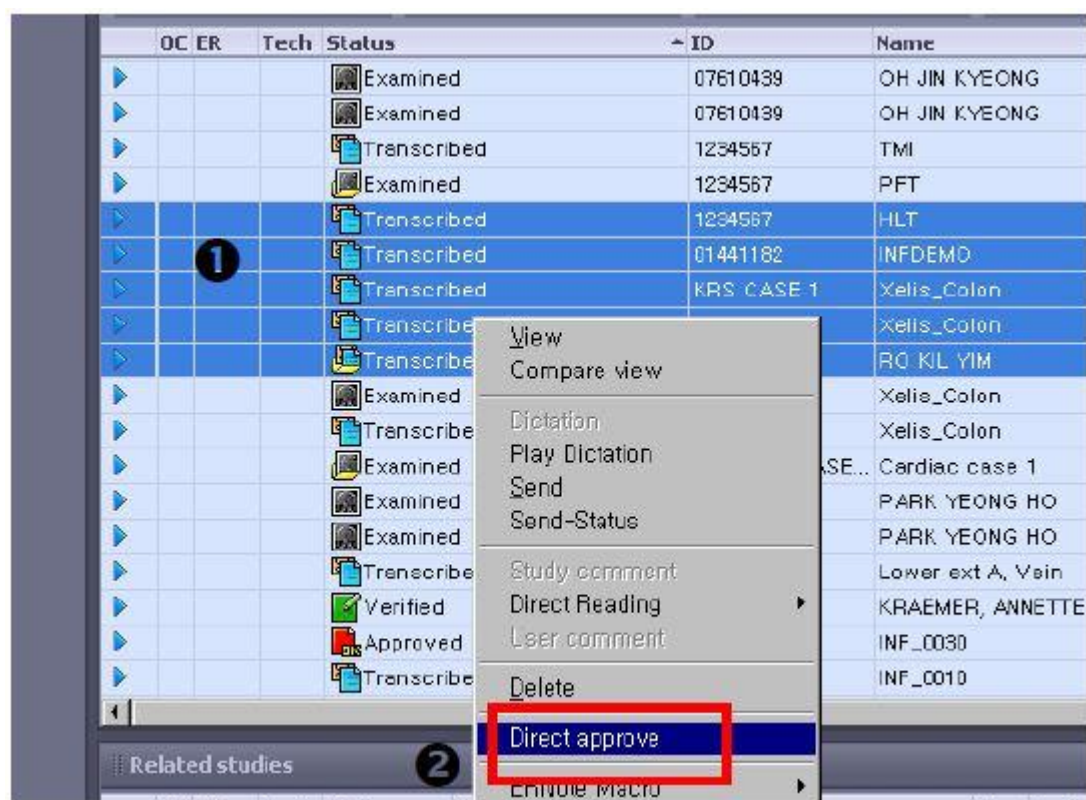
OC	ER	Tech	Status	ID	Name
			Examined	07610439	OH JIN KYEONG
			Examined	07610439	OH JIN KYEONG
			Transcribed	1234567	TMI
			Examined	1234567	PFT
			Transcribed	1234567	HLT
			Transcribed	01441182	INFDEMO
			Transcribed	KRS CASE 1	Xelis_Colon
			Transcribed		Xelis_Colon
			Transcribed		RO KIL YIM
			Examined		Xelis_Colon
			Transcribed		Xelis_Colon
			Examined	SE...	Cardiac case 1
			Examined		PARK YEONG HO
			Examined		PARK YEONG HO
			Transcribed		Lower ext A, Vein
			Verified		KRAEMER, ANNETTE
			Approved		INF_0030
			Transcribed		INF_0010

Context menu options: View, Compare view, Dictation, Play Dictation, Send, Send-Status, Study comment, Direct Reading, User comment, Delete, **Direct approve**, Enroll macro.

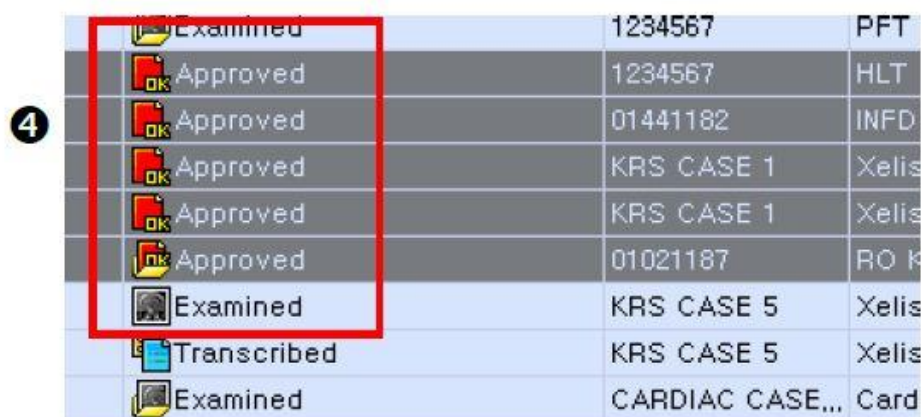
Related studies

Information

Do you want to approve all the selected studys?

OK Cancel

			Examined	1234567	PFT
			Approved	1234567	HLT
			Approved	01441182	INFDEMO
			Approved	KRS CASE 1	Xelis_Colon
			Approved	KRS CASE 1	Xelis_Colon
			Approved	01021187	RO KIL YIM
			Examined	KRS CASE 5	Xelis_Colon
			Transcribed	KRS CASE 5	Xelis_Colon
			Examined	CARDIAC CASE...	Cardiac case 1

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